

Changing a two-piece stoma pouch

Using the example of a urostomy
(same method applies for a colostomy and ileostomy).



OakMed 
Giving you more choice.
www.oakmed.co.uk

Dear Patient,

In hospital you had an artificial opening created for your bowel or bladder (a stoma).

It is important that you learn how to change your stoma pouch for yourself so that you can regain your independence as quickly as possible.

We would like to help you with this. Using the example of a urostomy (an artificial opening for the urinary tract), this booklet gives you a step-by-step guide to how to change your two-piece stoma pouch. Take all the time you need and do not be discouraged by minor setbacks if everything does not work out right away.

Your doctor or stoma-care nurse can answer specific questions you may have about your medication and care.



1. Lay out all the equipment you need to change your pouch. Remember adhesive remover spray if you use this.

2. After removing your pouch, clean and dry the stoma and the surrounding skin with the soft wipes provided, you can use warm tap water for this.





3. Cut the base plate of your pouch to fit the size of your stoma.

TIP

Where applicable, you can use a previously prepared template to help with this. Your stoma nurse will help you if you find this difficult.




4. Now remove the protective film from the base plate.



5. Now position the base plate exactly over your stoma. Check to ensure that it is correctly positioned and has fully adhered to the skin.

6. Now attach the bag to the base plate.

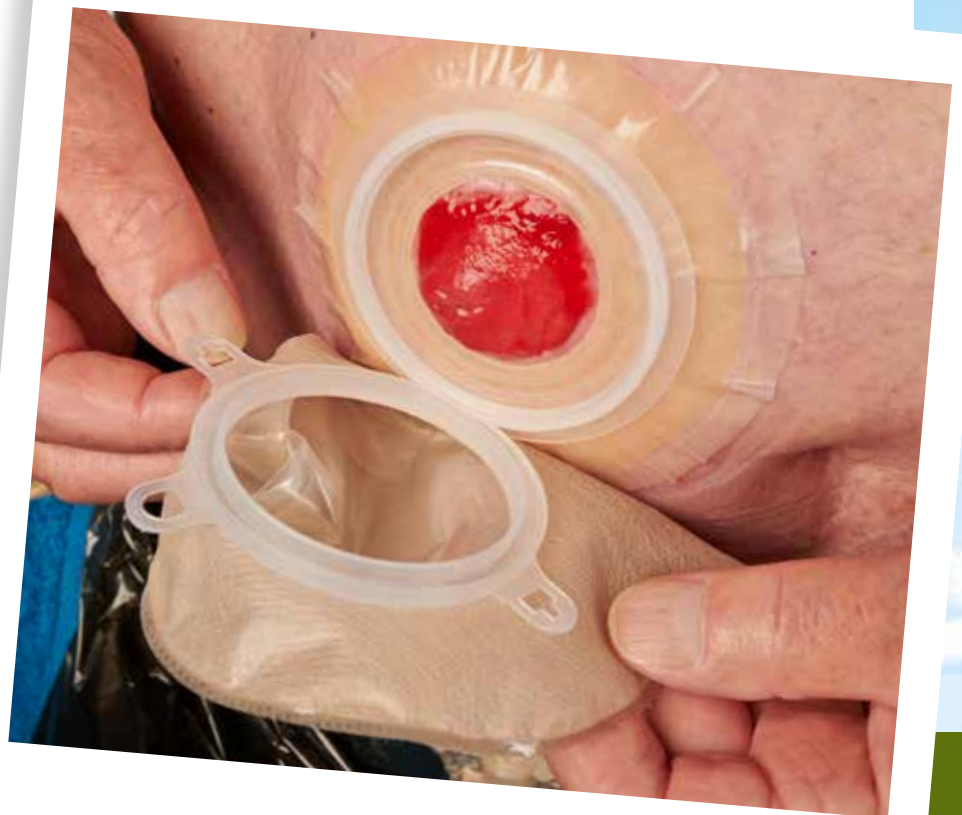



7. Check to ensure that the pouch is properly attached to the base plate.

8. Check to ensure that your clothing does not restrict your stoma pouch.

If you have any questions please contact your stoma nurse.

Stoma Nurse:

.....

Contact Number:

.....

Notes:

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....



To order samples, products or arrange home delivery services
please contact our customer service department on:

Freephone: 0800 592 786 | Main: 01604 586 529
Fax: 01604 584 784 | Email: orders@oakmed.co.uk



Oakmed Limited
Unit B, The I O Centre, Barn Way,
Lodge Farm Industrial Estate,
Northampton, NN5 7UW

OakMed 
Giving you more choice.

www.oakmed.co.uk