Changing a one-piece stoma pouch

Using the example of a urostomy

(same method applies for a colostomy and ileostomy).



Dear Patient,

In hospital you had an artificial opening created for your bowel or bladder (a stoma).

It is important that you learn how to change your stoma pouch for yourself so that you can regain your independence as quickly as possible. We would like to help you with this. Using the example of a urostomy (an artificial opening for the urinary tract), this booklet gives you a step-by-step guide to how to change your one-piece stoma pouch. Take all the time you need and do not be discouraged by minor setbacks if everything does not work out right away.

Your doctor or stoma-care nurse can answer any specific questions you may have about your medication and care.



2. After removing your pouch, clean and dry the stoma and the surrounding skin with the soft wipes provided. You can use warm tap water for this.








5. Position the opening of the bag against the lower edge of your stoma and apply it moving from the bottom to the top.

6. Then mould the adhesive area to your body, warming well.



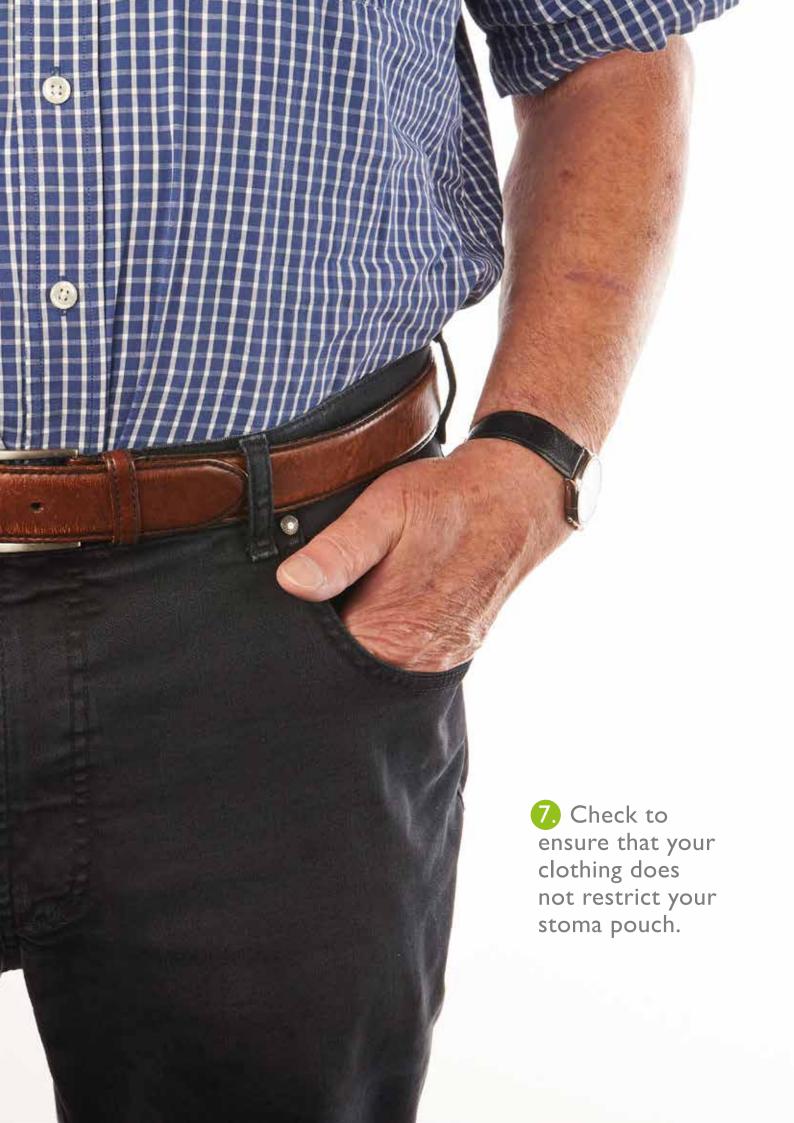

If you have any questions please contact your stoma nurse.

Stoma Nu	rse:
Contact N	Number:
Notes:	
• • • • • • • • • • • • • • • •	
•••••	
•••••	
• • • • • • • • • • • • • • • • • • •	,

To order samples, products or arrange home delivery services please contact our customer service department on:

Freephone: 0800 592 786 | Main: 01604 586 529 Fax: 01604 584 784 | Email: orders@oakmed.co.uk



Oakmed Limited
Unit B, The I O Centre, Barn Way,
Lodge Farm Industrial Estate,
Northampton, NN5 7UW



www.oakmed.co.uk