



OUR  
BEST HEALING  
FORMULA YET

Our Alginate wafer is one of our superior skin protectors offering both preventative and curative properties.



Anatomical shape on the closed bags



Highly absorbent adhesive



A unique wafer which promotes natural skin healing



A brand new closure



THINGS YOU NEED TO KNOW

15%  
Alginate  
in all products

Helps to  
heal  
sore  
peristomal skin

Reduces  
patient trauma  
& the need for  
accessories



“ After a week there was dramatic improvement in my skin condition ”

OUR SOFT CONVEX CLOSED BAG IS NOW AVAILABLE

Saves  
on nursing  
time and costs



High tack  
adhesion for advanced  
performance

Unique  
anatomical  
shape  
Is  
preventative  
and  
curative

For information or to speak to a member of our team about **Algininate**

Freephone 0800 592 786 or visit [www.oakmed.co.uk](http://www.oakmed.co.uk)

# What makes Alginate unique?

The wafer/flange on the Alginate products is unique, it incorporates seaweed which has been used for generations in wound care.

The wafer/flange is like using a dressing to sooth, heal and promote skin integrity as it has 15% active ingredients.

Get the most out of your Alginate bag by following these handy tips!



USE A HAIR DRYER

PLACE UNDER THE ARM

PLACE ON A RADIATOR

TUCK INTO YOUR WAISTBAND

## HOW TO APPLY FOR OUR BEST HEALING RESULTS




### STEP 1

Before applying the new pouch, **warm up to wake up** the healing properties within the wafer.



### STEP 3

Do not apply any powder, barrier film etc around the stoma (even if it is moist or bleeding) if you do, this will put a barrier between the skin and the healing properties in the wafer/flange (unless directed by a medical professional to do so).

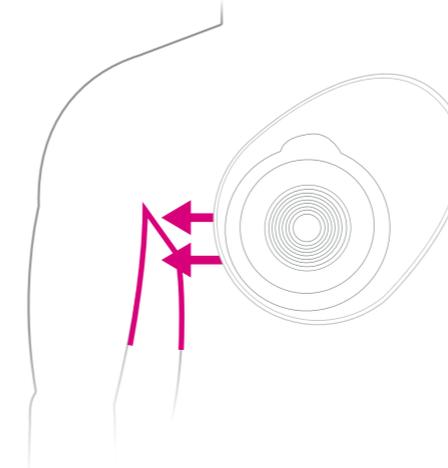

### STEP 2

This can be done in many ways, place under the arm, sit on it, put on a radiator, use a hairdryer or tuck into your waistband.

### STEP 4

When on, keep your hand over the new bag for a little longer.

